

**INSTALLATION INSTRUCTIONS**  
**SERIES 9140 3 Tiered Surgical Instrument Washer/Decontaminator**

**Specifications**

Series 9140 3 Tiered Surgical Instrument Washer/Decontaminator

Height	1540 mm
Width	660 mm
Depth (closed)	690 mm
Depth (open)	1320 mm

**Installation**

Hot and Cold stopcocks should be situated above or next to the machine either to left or right in an accessible position. Top of standing waste to be 1000mm from floor. Drain outlet hose to be 400mm above bottom of door.

**Service Requirements**

Water Supply	Hot Water: 1 inlet valve for cold water 20 mm BSP 50kPa min
	Cold Water: 1 inlet valve for hot water 20 mm BSP 50kPa min
	Temperature of hot water supply 60°C
	2 Inlet hoses have 3/4" internal thread
	Pex inlet hose length 165 cm

Electrical	240V, 50Hz 25Amps Machine equipped with flexible tail for connection to isolating switch
------------	---

Drainage	38mm standing waste or connection nipple on the waste trap of the sink.
----------	---

**Drainage Connection**

The machine is supplied with the outlet hose connected. Run the outlet hose to connection tee on the 'P' trap or 'S' trap on an adjoining sink waste or standard pipe.

**Note:**

- The hose must always discharge at least 1m above the floor.
- The hose must not extend by more than 3m. joints and extension pipes must have an internal diameter of not less than 18mm (3/4")
- The hose must not be run directly to a floor drain or similar outlet. If this is done, it can act as a siphon and empty the machine.

